GB 1. Intended use / Indication

US Light-curable polymerizable resin intended to be used in conjunction with curing light equipment. Indicated for the fabrication of earmolds.

2. Contraindication

audioprint® GR-10.2 is contraindicated ...

1. ... If a patient is known to be allergic to any of the ingredients.
2. ... for every application that is not part of the indication (see above).

3. Patient Target Group

Persons for whom an earmold is to be created.

4. Intended Users

Earmold laboratory, ENT specialist, hearing aid acoustician

5. Requirements

Software – Information available from:

Cyflex AG · Binzmühlestrasse 15, 8050 Zürich, Switzerland
3SHAPE A/S · Holmens Kanal 7, 1060 Copenhagen, Denmark

Hardware (3D Printing) – Information available from:

ASIGA · 2/19-21 Bourke Road, Alexandria, NSW 2015, Australia

Hardware (Post Curing) – Information available from:

NK Optik GmbH (otoflash G171) · Isarstr. 2, D-82065 Baierbrunn, Germany

Additional specifications on www.pro3dure.com.

6. Material

audioprint® GR-10.2 consists of functional (meth)acrylic resins, initiators, dyes and stabilisers.

7. Geometric presetting

According to the appropriate design specifications (templates).

8. Material parameters

Radiation penetration depth controllable by exposure time

50 µm

100 µm

9. Manufacturing process (fig. 1–10)

1. Prepare data (CAD & build preparation).
2. Choose process parameters (build style, etc.).
3. Transfer prepared data to 3D printing device.
4. Prepare 3D printing – shake bottle.
5. Fill resin tank of 3D printing device.
6. Build the parts.
7. Clean parts (with IPA ≥ 97 % or equivalent cleaning agent) approx. 4 min. in an ultrasonic bath or equivalent device – precleaning recommended.
8. Dry parts, until there are no residues of IPA or equivalent cleaning agent.
9. Post curing (2 x 4000 flashes - Otofash G171): Inert atmosphere recommended.
10. Finish parts.

10. Finishing processes

Polishing

Coating (audioprint® L-1 recommended)

11. Notice

Please follow the instructions provided by the software manufacturer in terms of parameter settings and design recommendations. Please follow the instructions provided by the hardware manufacturer in terms of parameter settings/printing & post curing recommendations. To avoid detrimental effects on material quality do not expose the liquid material to irradiation under any circumstances. Deviations from the described manufacturing processes or storing conditions may lead to different mechanical and optical properties of the material. According to the EU Medical Devices Regulation, users/patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred. **Caution:** The lot number and the best before date are indicated on each packaging. In case of claims please always indicate the lot number of the product. Do not use the product after expiry of the best before date. Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Avoid breathing dust/fume/gas/mist/vapours/spray. Avoid contact during pregnancy/while nursing. Wash hands thoroughly after handling. Contaminated work clothing should not be allowed out of the workplace. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If skin irritation occurs: Get medical advice/attention. Keep container tightly closed. If skin irritation or rash occurs: Get medical advice/attention. If eye irritation persists: Get medical advice/attention. Take off contaminated clothing and wash it before reuse. Store in a well-ventilated place. Keep container tightly closed. Store locked up. Dispose of contents/container according to official regulations.

12. Hazard statements

Causes skin irritation. May cause an allergic skin reaction. Causes serious eye irritation. Suspected of damaging the unborn child. Toxic to aquatic life with long lasting effects.

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