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Distributor (US):
pro3dure medical LLC
9825 Valley View Road · Eden Prairie, MN 55344
Phone: 952-426-1928

info@pro3dure.com · www.pro3dure.com · Made in Germany

1. Intended use / Indication

Methacrylate based resin for 3D-printing systems (DLP) with irradiation sources (385 nm, 405 nm) for the production of cast forms.

2. Contraindications

audioprint® GR-12.3 is contraindicated ...

- ... if a user is known to be allergic to any of the ingredients.
- ... for direct contact with uncured or partially cured material.
- ... for any application that is not part of the indication (see above).

3. Intended users

Earmold laboratory, hearing aid acoustician

4. Requirements

Software – Information available from:

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Hardware (3D Printing) – Information available from:

ASIGA · 2/19-21 Bourke Road, Alexandria, NSW 2015, Australia

Hardware (Post Curing) – Information available from:

NK Optik GmbH (otoflash G171) · Isarstr. 2, D-82065 Baierbrunn, Germany

Additional specifications on www.pro3dure.com.

5. Material

audioprint® GR-12.3 consists of functional (meth)acrylic resins, initiators, dyes and stabilisers.

6. Geometric presetting

Minimum wall thickness: 0.5 mm

7. Material parameters

Radiation penetration depth controllable by exposure time

50 µm

100 µm

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1) audioprint® GR-12.3 bottles should be shaken well before use. Ensure that audioprint® GR-12.3 is tempered to 23 °C to 30 °C before use.

2) Carefully pour audioprint® GR-12.3 into the specified printer tray of the production system.

3) Ensure that the correct material parameters are selected on the printer.

4) Print molds from audioprint® GR-12.3. Ensure that the vent holes are level with the filler neck. Direct post-processing is recommended after completion of the build process. If this cannot be guaranteed, leave the produced objects in the printing system.

5) After printing, clean the parts thoroughly with IPA in a 5-minute ultrasonic bath and then rinse them with fresh IPA.

6) Dry the parts thoroughly inside and out with compressed air.

7) Curing with Otofash G171: Cure the parts with 4000 flashes in an Otofash G171, under nitrogen.

9. Notice

The batch number and best-before date are indicated on each package. In case of complaints, please always state the batch number of the product. Do not use the product after the expiration date. Deviations from the described manufacturing processes or storage conditions may lead to changes in the mechanical and optical properties of the material. Due to its chemical base, audioprint® GR-12.3 is sensitive to moisture and should be stored in a dry place at room temperature.

10. Hazard Statements:

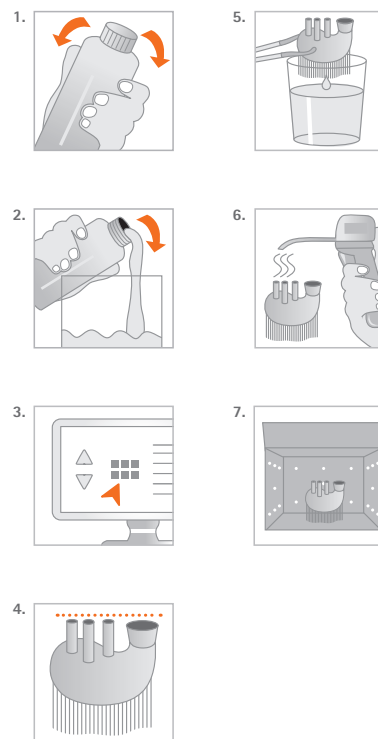
May cause an allergic skin reaction.

11. Precautionary Statements:

Avoid breathing dust/fume/gas/mist/vapours/spray. Wash hands thoroughly after handling. Contaminated work clothing should not be allowed out of the workplace. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash it before reuse. Store in a well-ventilated place. Keep container tightly closed. Store locked up. Dispose of contents/container according to official regulations.

**Generative Resin
GR-12.3**

Instructions for use

Product compatibility

	SysTherm® ECO	SysTherm®
SysTherm® cast 2.0	SysTherm® SPT-1 or SPT-2	SysTherm® SPT-1 or SPT-2

Physical properties*:

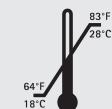
audioprint® GR-12.3

■ Density:
approx. 1.1 g/ml

■ Viscosity:
approx. 0,5 Pa s

■ Cured material:
(depending on irradiation unit)
Modulus of elasticity:
approx. 700 MPa
Flexural strength:
approx. 30 MPa
Elongation at break:
approx. 15 %

■ Storage:

Ordering information:

audioprint® GR-12.3

1 kg:
 $\lambda \leq 405 \text{ nm}$

yellow-transparent

REF: A1001004

* These data come from measurements of a representative sample that was determined as part of our quality assurance.

This side up.
Diese Seite nach oben.
Ce côté vers le haut.
Este lado hacia arriba.
Questo lato in su.

Temperature limit
Temperaturgrenze
Limite de température
Limite de temperatura
Limite di temperatura

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pro3dure medical GmbH