printo[®] dent

Generative Resin GR-13.2 model

Instructions for use
Gebrauchsanweisung
Mode d'emploi
Instrucciones de uso
Istruzioni per l'uso

pro3d[®]ure medical

Materials are our DNA!

1. Intended use / Indication

Light-curable polymerizable resin intended to be used in conjunction with extra-oral curing light equipment for the fabrication, by additive manufacturing, of dental appliances. Indicated for the production of dental models.

2. Contraindication

printo[®] GR-13.2 model is contraindicated for any use on patients (printo[®] GR-13.2 model is only a laboratory product).

3. Intended Users

Dentist, dental technicians

4. Requirements

Software – information available from:

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Hardware (3D Printing) – information available from:

ASIGA · 2/19-21 Bourke Road, Alexandria, NSW 2015, Australia

Hardware (Post Curing) – information available from:

pro3dure medical (CD-1, CD-2) · Am Burgberg 13, D-58642 Iserlohn, Germany

Additional specifications on www.pro3dure.com.

5. Material

printo[®] GR-13.2 model consists of functional (meth)acrylic resins, initiators, dyes and stabilisers.

6. Geometric presetting

According to the design specifications of the applicable CAD software.

7. Material parameters

Radiation penetration depth controllable by exposure time:

50 µm

100 µm

8. Manufacturing process (fig. 1-10)

1. Prepare data (CAD & build preparation).
2. Choose process parameters (build style, etc.).
3. Transfer prepared data to 3D printing device.
4. Prepare 3D printing – shake bottle.
5. Fill resin tank of 3D printing device.
6. Build the parts.
7. Clean parts (with IPA ≥ 97 % or equivalent cleaning agent) approx. 4 min. in an ultrasonic bath or equivalent device – precleaning recommended.
8. Dry parts, until there are no residues of IPA or equivalent cleaning agent.
9. Post curing (4 min.): inert atmosphere recommended (use adequate light curing device).
10. Finish parts.

9. Finishing processes

According to the further uses (e.g. insulating, blocking out, etc.).


10. Notice

Please follow the instructions provided by the software manufacturer in terms of parameter settings and design recommendations. Please follow the instructions provided by the hardware manufacturer in terms of parameter settings/printing & post curing recommendations. To avoid detrimental effects on material quality do not expose the liquid material to irradiation under any circumstances. Deviations from the described manufacturing processes or storing conditions may lead to different mechanical and optical properties of the material. **Caution:** The lot number and the best before date are indicated on each packaging. In case of claims please always indicate the lot number of the product. Do not use the product after expiry of the best before date. Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Avoid breathing dust/fume/gas/mist/vapours/spray. Avoid contact during pregnancy/while nursing. Wash hands thoroughly after handling. Contaminated work clothing should not be allowed out of the workplace. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If skin irritation or rash occurs: Get medical advice/attention. If eye irritation persists: Get medical advice/attention. Take off contaminated clothing and wash it before reuse. Store in a well-ventilated place. Keep container tightly closed. Store locked up. Dispose of contents/container according to official regulations.

11. Hazard statements

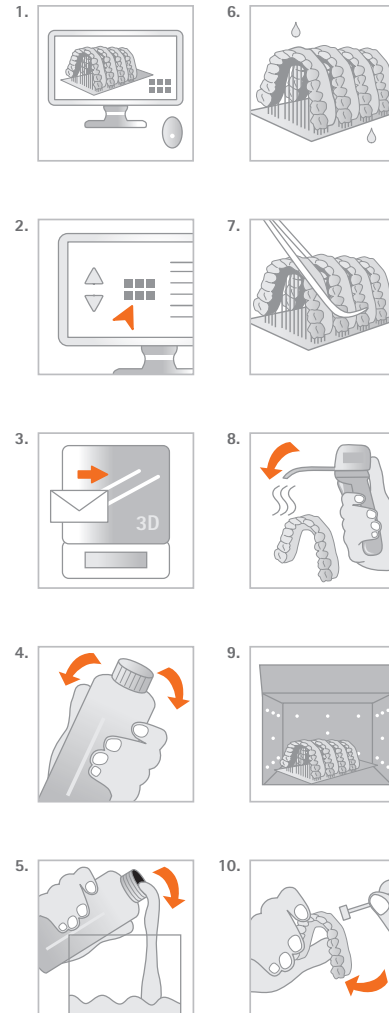May cause an allergic skin reaction. Causes serious eye irritation. May damage fertility. Suspected of damaging the unborn child.

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