1. Indication
printo[®] GR-22 flex from pro3dure is intended for the production of orthodontic and dental objects (mouthguards, nightguards, snoring appliances and splints).

2. Contraindication

printo[®] GR-22 flex is contraindicated ...

1. ... if a patient is known to be allergic to any of the ingredients.
2. ... for denture bases.
3. ... for every application that is not part of the indication (see above).

3. Patient Target Group

Persons being treated in the context of a dental procedure.

4. Intended Users

Dentist, dental technician

5. Requirements

Software – information available from:

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Hardware (3D Printing) – information available from:

ASIGA · 2/19-21 Bourke Road, Alexandria, NSW 2015, Australia

Hardware (Post Curing) – Information available from:

NK Optik GmbH (otoflash G171) · Isarstr. 2, D-82065 Baierbrunn, Germany

Additional specifications on www.pro3dure.com.

6. Material

printo[®] GR-22 flex consists of functional (meth)acrylic resins, initiators, dyes and stabilisers.

7. Geometric presetting

Minimum wall thickness: 1.5 mm

8. Material parameters

Radiation penetration depth controllable by exposure time

50 µm

100 µm

9. Manufacturing process (fig. 1-10)

1. Prepare data (CAD & build preparation).
2. Choose process parameters (build style, etc.).
3. Transfer prepared data to 3D printing device.
4. Prepare 3D printing – shake bottle.
5. Fill resin tank of 3D printing device.
6. Build the parts.
7. Clean parts (with IPA ≥ 97 % or equivalent cleaning agent) approx. 10 min. in an ultrasonic bath or equivalent device – precleaning recommended.
8. Dry parts, until there are no residues of IPA or equivalent cleaning agent.
9. Post curing (2 x 2500 flashes – otoflash G171): inert atmosphere recommended.
10. Finish parts.

10. Finishing processes

Polishing

11. Notice

Please follow the instructions provided by the software manufacturer in terms of parameter settings and design recommendations. Please follow the instructions provided by the hardware manufacturer in terms of parameter settings/printing & post curing recommendations. To avoid detrimental effects on material quality do not expose the liquid material to irradiation under any circumstances. Deviations from the described manufacturing processes or storing conditions may lead to different mechanical and optical properties of the material. According to the EU Medical Devices Regulation, users/patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred. **Caution:** The lot number and the best before date are indicated on each packaging. In case of claims please always indicate the lot number of the product. Do not use the product after expiry of the best before date. Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Avoid breathing dust/fume/gas/mist/vapours/spray. Wash hands thoroughly after handling. Use only outdoors or in a well-ventilated area. Contaminated work clothing should not be allowed out of the workplace. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. IF INHALED: Remove person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. IF exposed or concerned: Call a POISON CENTER/doctor. Call a POISON CENTER or doctor/physician if you feel unwell. Specific treatment (see information on this label). If skin irritation occurs: Get medical advice/attention. If skin irritation or rash occurs: Get medical advice/attention. If eye irritation persists: Get medical advice/attention. Take off contaminated clothing and wash it before reuse. Collect spillage. Store in a well-ventilated place. Keep container tightly closed. Store locked up. Dispose of contents/container according to official regulations.

12. Hazard statements

Causes skin irritation. May cause an allergic skin reaction. Causes serious eye irritation. May cause respiratory irritation. Suspected of damaging fertility or the unborn child. Toxic to aquatic life with long lasting effects.

Generative Resin

GR-22 flex

Instructions for use · Gebrauchsanweisung
Mode d'emploi · Instrucciones de uso
Istruzioni per l'uso

Physical properties*/
Physikalische Eigenschaften*/
Propriétés physiques*/
Propiedades físicas*/
Proprietà fisiche*:

printo[®] GR-22 flex

- Shore hardness D/
Shore-Härte D/
Dureté Shore D/
Durezza Shore D
Durezza Shore D
(ISO 48-4):
≥ 65**
- Ultimate flexural strength/
Biegefestigkeit/
Résistance à la flexion/
Fuerza flexible/
Resistenza alla flessione
MPa (ISO 20795-2):
≥ 2**
- Bending module/
Biegemodul/
Module de pliage/
Módulo de flexión/
Modulo di piegatura
MPa (ISO 20795-2):
≥ 100**
- Water absorption/
Wasseraufnahme/
Absorption de l'eau/
Absorción de agua/
Assorbimento dell'acqua
µg/mm³ (ISO 20795-2):
≤ 32
- Solubility/
Löslichkeit/
Solubilité/
Solubilidad/
Solubilità
µg/mm³ (ISO 20795-2):
≤ 5
- Impact strength
(IZOD notched, ASTM D256)/
Schlagfestigkeit
(IZOD notched, ASTM D256)/
Résistance aux chocs
(IZOD entaillé, ASTM D256)/
Resistencia al impacto
(IZOD con muescas, ASTM D256)/
Resistenza agli urti
(IZOD con intaglio, ASTM D256):
≥ 50 J/m**
- Elongation at break (ISO 37)/
Reißdehnung (ISO 37)/
Allongement à la rupture (ISO 37)/
Alargamiento a la rotura (ISO 37)/
Allungamento a rottura (ISO 37):
≥ 40 %**

Ordering information/
Bestellinformationen/
Informations de commande/
Información sobre pedidos/
Informazioni sull'ordine:

printo[®] GR-22 flex

1 kg:
λ ≤ 405 nm

clear-transparent/
klar-transparent/
incoloro-transparente/
incoloro-transparente/
incoloro-transparente
REF: D1001701

light blue/
hellblau/
bleu clair/
azul claro/
azzurro
REF: D1001703

500 g:
λ ≤ 405 nm

clear-transparent/
klar-transparent/
incoloro-transparente/
incoloro-transparente/
incoloro-transparente
REF: D1001702

light blue/
hellblau/
bleu clair/
azul claro/
azzurro
REF: D1001704

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